



Jacoti ListenApp

User Guide


for Jacoti ListenApp Version 1.0
revision August 20th, 2013

What does Jacoti ListenApp do?


Jacoti ListenApp is a software hearing aid that amplifies sound and hereby compensates for impaired hearing. It is indicated for mild to moderate hearing losses. You need an audiogram that has been measured by a hearing professional. The audiogram documents your individual hearing loss and is used by Jacoti ListenApp to compensate for your impaired hearing: Jacoti ListenApp amplifies some frequencies more than others. Jacoti ListenApp also amplifies soft sounds more than loud sounds and it detects and reduces noise signals. You can use Jacoti ListenApp with an iPhone 4 / 4S / 5 and with an iPod Touch 4th or 5th generation. The version of iOS must be iOS 6.0 or higher.



You should use the original Apple EarPods® with remote and microphone that allow you to adjust the sound level in the remote. Note that the microphone can pick up undesired sounds that are caused by friction with your clothes. You can also change the volume of the sound with the hardware buttons on the side of the device.

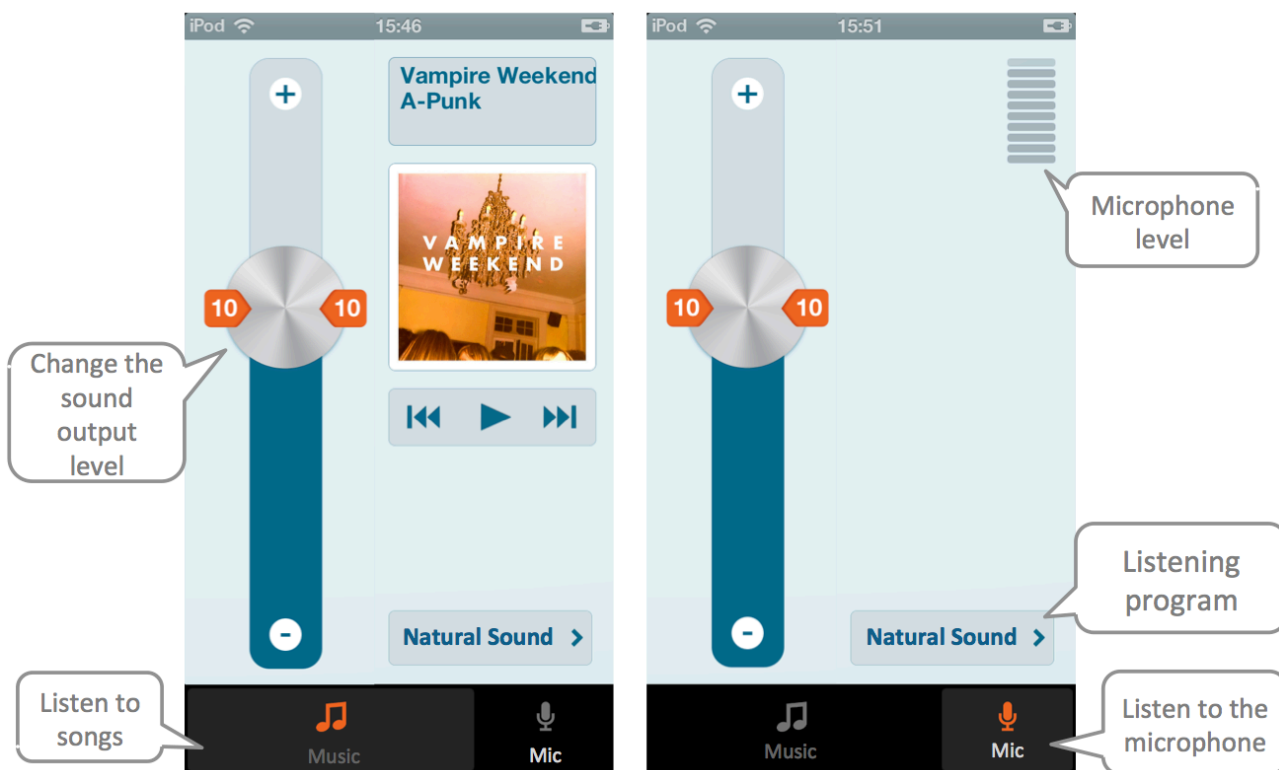

The main screen of Jacoti ListenApp sports a large volume slider that allows you to easily adjust the sound volume. The tab at the bottom of the screen allows you to switch between listening to music and the microphone. Pressing the listening program button will take you to the Program screen shown hereafter.

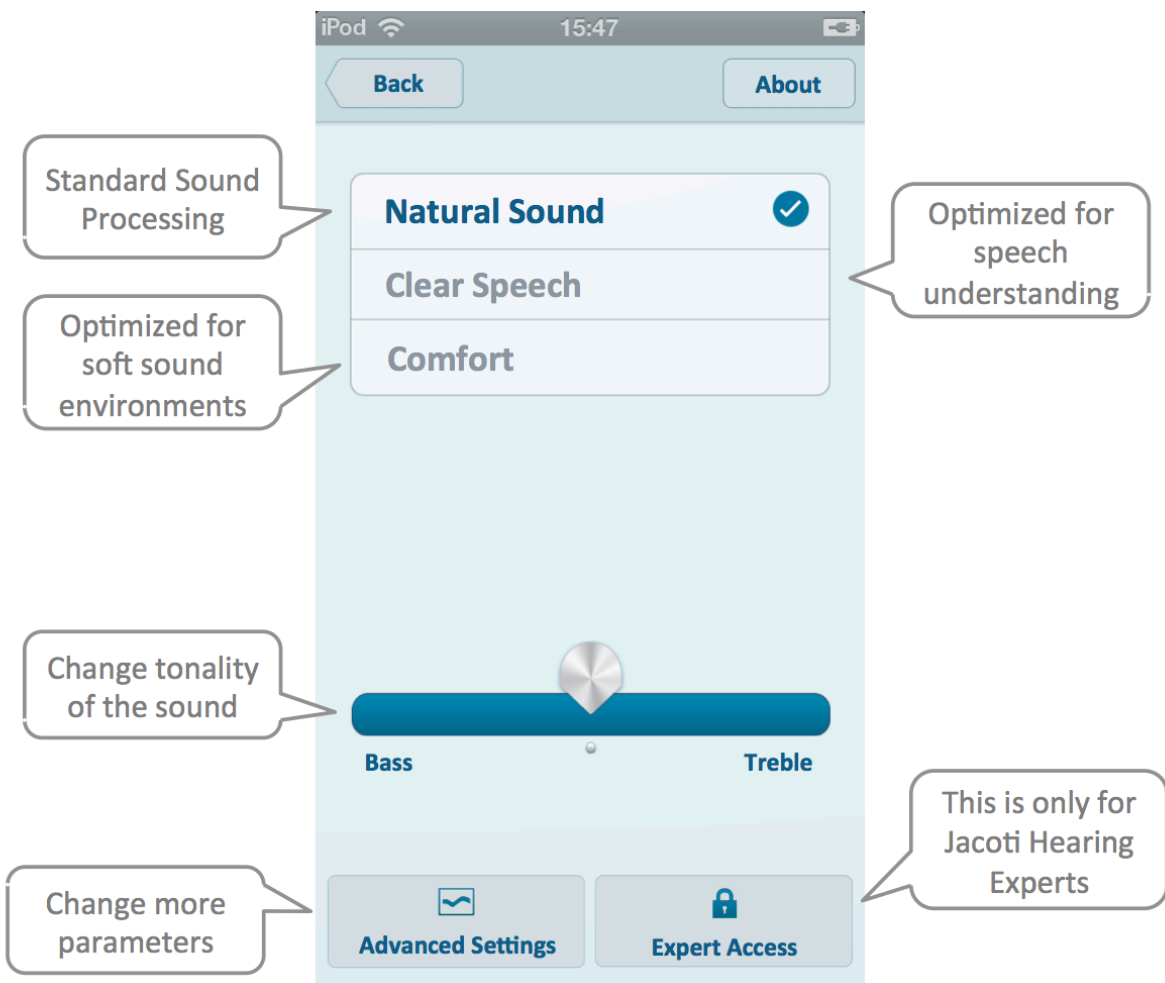

The Program screen allows you to change the tonality of the sound and to select a listening program for special situations.

In the listening program “Natural Sound”, Jacoti ListenApp compensates for your hearing loss, and makes no assumptions about the listening situation. The “Clear Speech” listening program has been optimized specifically for speech understanding. This is a listening program that can be helpful during meetings or presentations, to watch TV, or when chatting with friends and family. The comfort listening program provides a bit less amplification to reduce fatigue, for example when you are driving a car. In the Music mode, Jacoti ListenApp also offers the listening program “Classical Music” which provides more amplification to soft parts of music pieces. However, the “Natural Sound” listening program is our recommendation for most songs.

Getting help from a hearing professional

You can ask a hearing professional for help to find better settings of Jacoti ListenApp. The hearing professional can visit the web page <https://restricted.jacoti.com> to become a Jacoti Hearing Expert. He can then access more parameters in Jacoti ListenApp and he can also change parameters remotely over the Internet.

Declaration of Conformity

This Jacoti product meets the requirements of Annex II of the Medical Device Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from the manufacturer.

Trademarks

Jacoti® and Jacoti ListenApp® are registered trademarks of Jacoti bvba.

Warnings



Jacoti ListenApp should only be used with original Apple EarPods headphones. On the new 5th gen iPod Touch, you have to use the Apple EarPods with remote and microphone. If you have hearing aids, do not use Jacoti ListenApp together with your hearing aids.



Partial occlusion of the human ear canal by the EarPods headphones can lead to an irritation of the outer ear. In this case, please consider reducing the usage of Jacoti ListenApp and consult your local ENT doctor.



In order to use Jacoti ListenApp for many hours, keep your battery charged.



Jacoti ListenApp cannot amplify sound during a GSM phone call.



Jacoti ListenApp version 1.0 can be safely used until October 2014. After this date, this version of Jacoti ListenApp is no longer supported by Jacoti bvba. Check for updates, a newer version of Jacoti ListenApp might be available in the app store.



Only use Jacoti ListenApp if you have a recent audiogram and a hearing loss within the fitting range of ListenApp.



You can find our privacy policy, trademarks and intellectual property on our web page at <http://www.jacoti.com/>



If you have a jailbroken phone, Jacoti ListenApp is not supported.

Manufacturer



Jacoti bvba
Dr. Theo Tutsstraat 45
B 2530 Boechout
Belgium



You can contact us at info@jacoti.com